


# Fixed/Removable Prosthesis

Number of Cases	718
-----------------	-----


Fixed/Removable	Number	Percentage
Fixed	686	95.5%
Removable	29	4.0%
Remaining Cases	3	0.4%

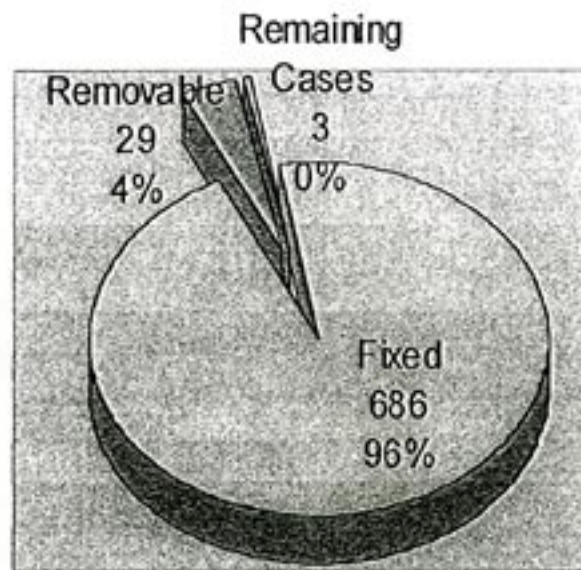

# Bone Density Types

Number of Implants	1623
--------------------	------


Bone Density Types	Number	Percentage
I	88	5.4%
II	894	55.1%
III	524	32.3%
IV	96	5.9%
Remaining Implants	21	1.3%

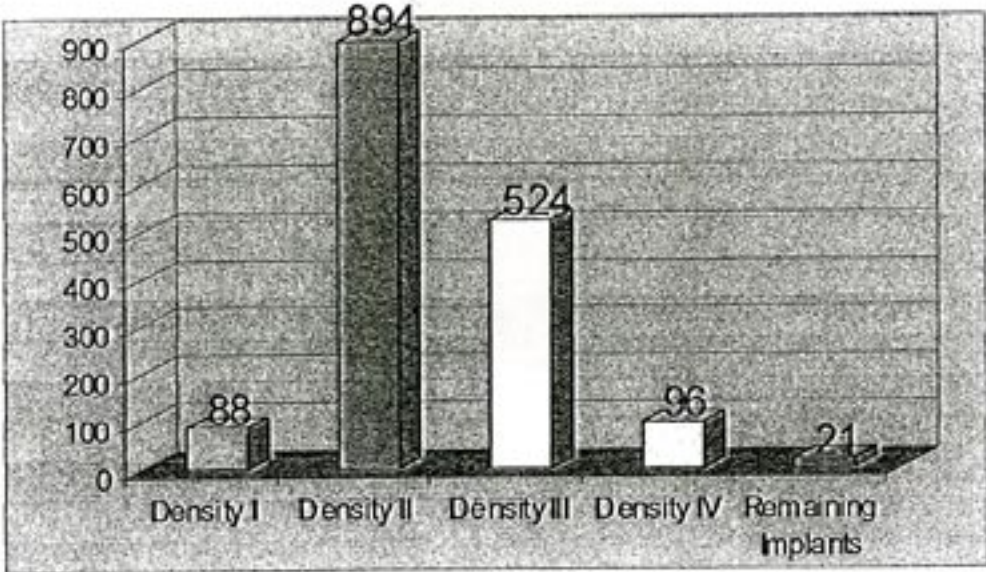

# Bone Volume Types

Number of Implants	1623
--------------------	------


Bone Volume Types	Number	Percentage
A	545	33.6%
B	739	45.5%
C	315	19.4%
D	5	0.3%
Remaining Implants	19	1.2%

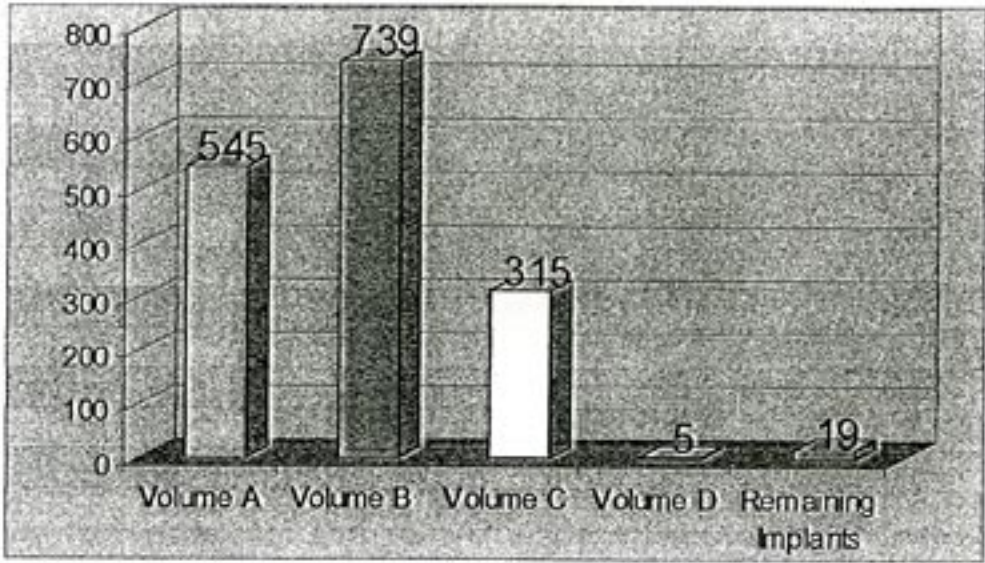

## Crest Types

Number of Implants	1623
--------------------	------


Crest Types	Number	Percentage
Edentulous Crest	1088	67.0%
Extraction/Implantation	517	31.9%
Remaining Implants	18	1.1%

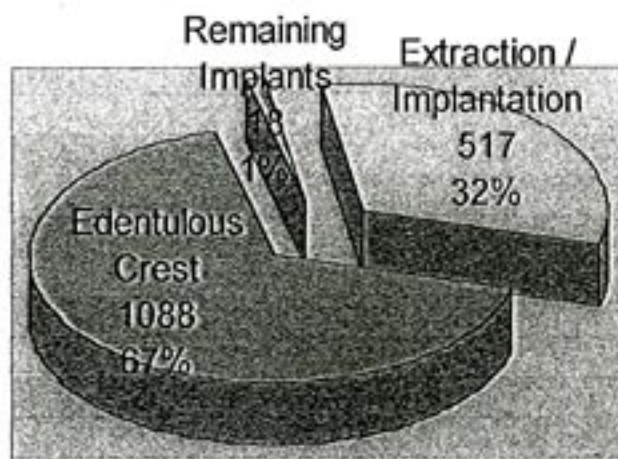

# Occlusion Types

Number of Cases	718
-----------------	-----

Occlusion Types	Number	Percentage
Normal	707	98.5%
Overjet	6	0.8%
Edge to Edge	5	0.7%
Remaining Cases	0	0.0%

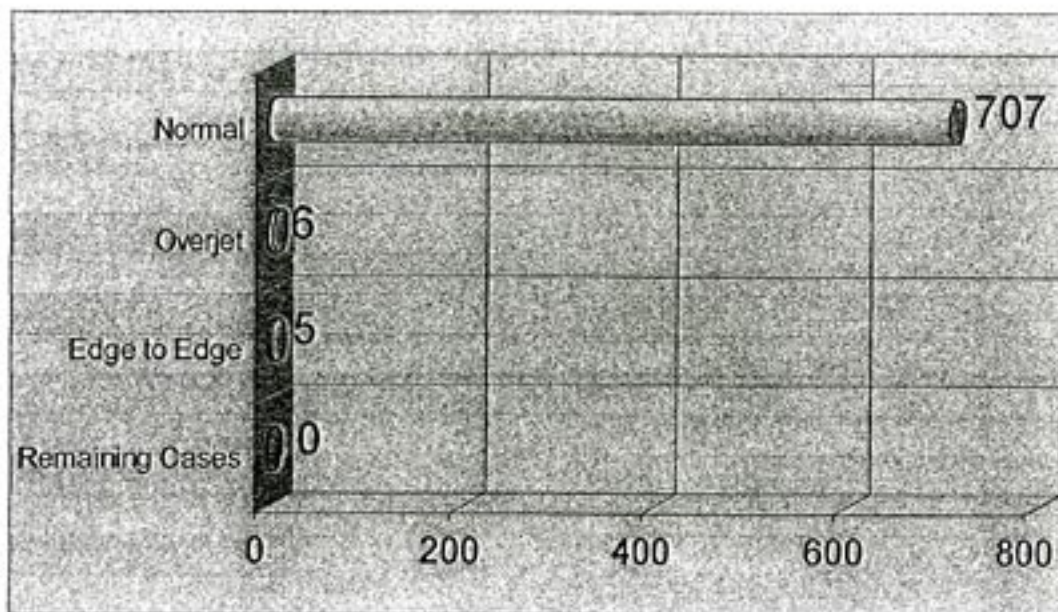


## Patient Informations

Number of Cases	718
-----------------	-----

Average Age	60.1
-------------	------

Sex	Number	Percentage
Men	274	38.2%
Women	444	61.8%
Remaining Cases	0	0.0%

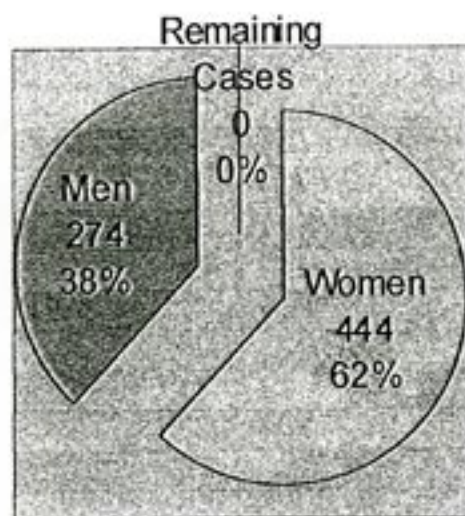

## Implants per Site

	Percentage	Number	Number	Percentage	
Site 11	2.3%	37	46	2.8%	Site 21
Site 12	3.0%	49	41	2.5%	Site 22
Site 13	3.8%	62	37	2.3%	Site 23
Site 14	6.0%	97	83	5.1%	Site 24
Site 15	5.7%	93	79	4.9%	Site 25
Site 16	2.2%	36	44	2.7%	Site 26
Site 17	1.4%	22	17	1.0%	Site 27
Site 18	0.9%	15	19	1.2%	Site 28
Site 41	0.6%	9	16	1.0%	Site 31
Site 42	1.6%	26	20	1.2%	Site 32
Site 43	2.2%	35	40	2.5%	Site 33
Site 44	3.3%	54	44	2.7%	Site 34
Site 45	4.3%	70	71	4.4%	Site 35
Site 46	8.1%	132	144	8.9%	Site 36
Site 47	5.2%	85	92	5.7%	Site 37
Site 48	0.2%	4	4	0.2%	Site 38

**Total**      1623


## Number of Implants: 4,00mm Diameter

Total Number of Implants	1623
--------------------------	------

Number of Unknown References	12	0.7%
------------------------------	----	------

Number of 4,00mm Implants	610	49.9%
---------------------------	-----	-------

4,00mm References	Number	Percentage
1020	33	2.0%
1021	67	4.1%
1022	155	9.6%
1023	118	7.3%
1024	241	14.8%
1025	196	12.1%

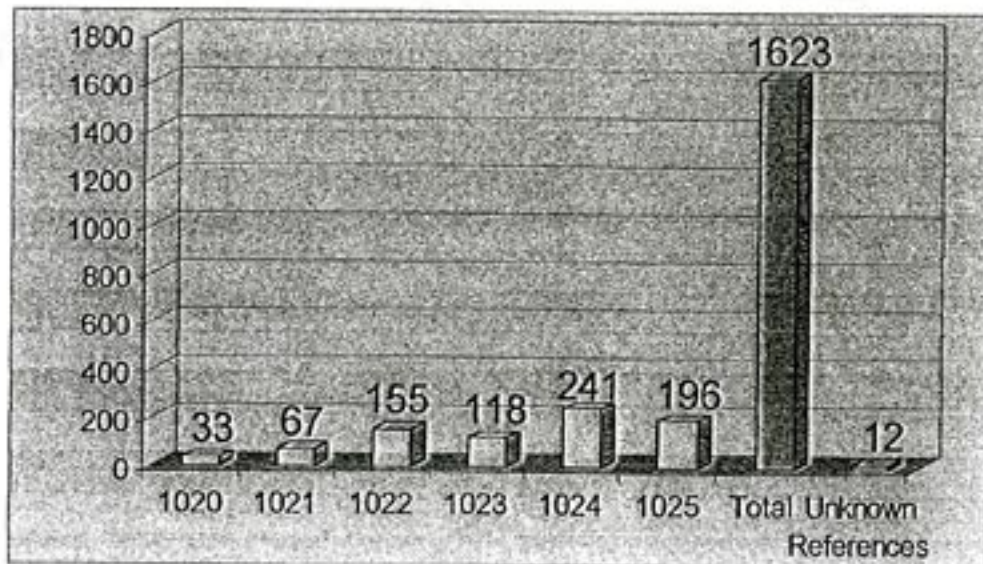


## Number of Implants: 4,75mm Diameter

Total Number of Implants	1623
--------------------------	------

Number of Unknown References	12	0.7%
------------------------------	----	------

Number of 4,75mm Implants	609	37.5%
---------------------------	-----	-------

4,75mm References	Number	Percentage
1030	56	3.5%
1031	90	5.5%
1032	216	13.3%
1033	180	11.1%
1034	67	4.1%

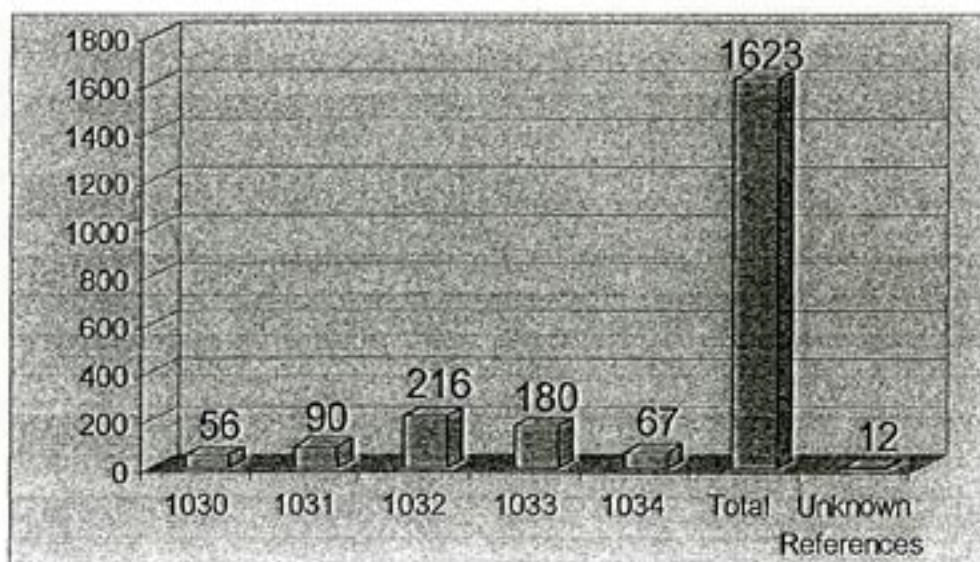


## Number of Implants: 6,00mm Diameter

Total Number of Implants	1623
--------------------------	------

Number of Unknown References	12	0.7%
------------------------------	----	------


Number of 6,00mm Implants	192	11.8%
---------------------------	-----	-------

6,00mm References	Number	Percentage
1040	22	1.4%
1041	43	2.6%
1042	85	5.2%
1043	38	2.3%
1044	4	0.2%

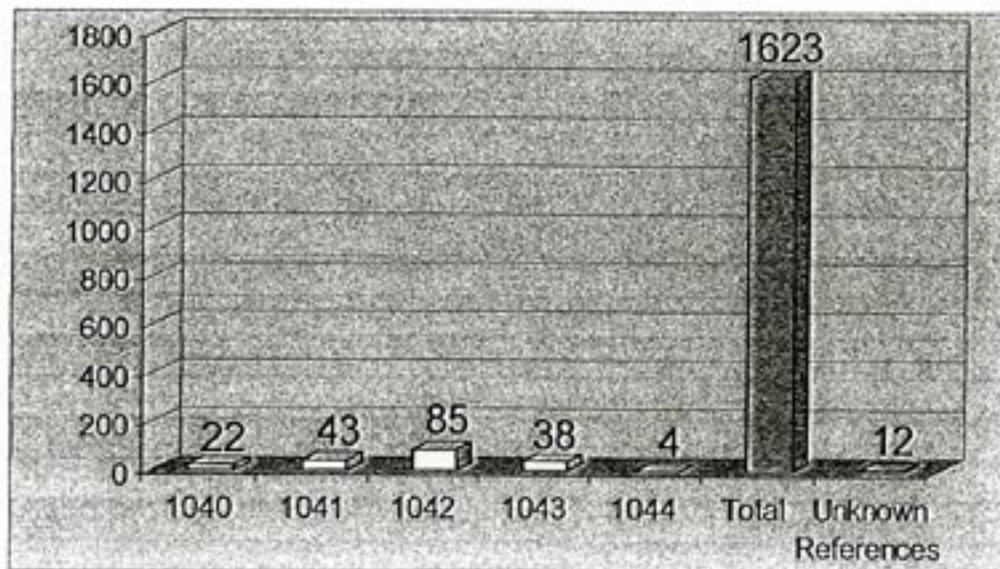

## Immediate/Delayed Loading


Number of Implants	1623	
Immediate/Delayed	Number	Percentage
Immediate	897	55.3%
Delayed	719	44.3%
Remaining Implants	7	0.4%

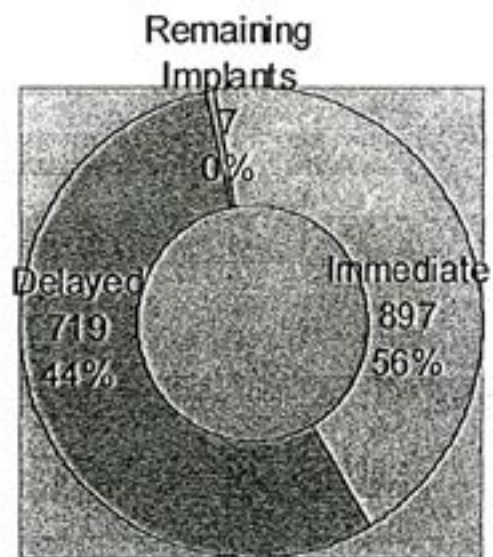

## Single Unit/Multiple Units Loading

Number of Implants	1623
--------------------	------

Single Unit/Multiple Units	Number	Percentage
Single Unit	845	52.1%
Multiple Units	757	46.6%
Remaining Implants	21	1.3%

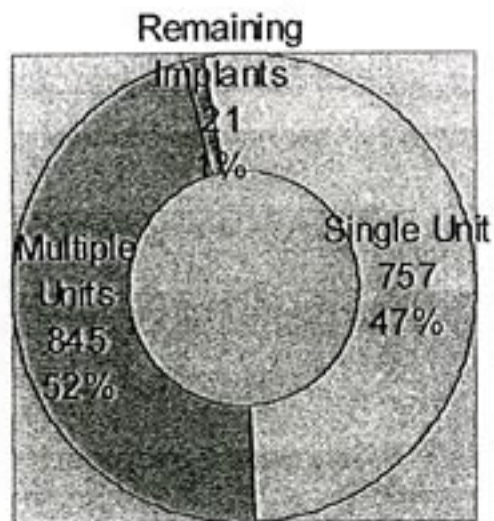

## Average ISQ at 6 weeks per Site

Average ISQ: 11	65.1	65.7	Average ISQ: 21
Average ISQ: 12	66.7	65.7	Average ISQ: 22
Average ISQ: 13	66.6	66.8	Average ISQ: 23
Average ISQ: 14	66.0	66.3	Average ISQ: 24
Average ISQ: 15	66.6	68.2	Average ISQ: 25
Average ISQ: 16	68.7	67.5	Average ISQ: 26
Average ISQ: 17	66.6	63.8	Average ISQ: 27
Average ISQ: 18	59.3	60.3	Average ISQ: 28
Average ISQ: 41	70.6	71.2	Average ISQ: 31
Average ISQ: 42	70.7	69.1	Average ISQ: 32
Average ISQ: 43	70.0	69.6	Average ISQ: 33
Average ISQ: 44	69.8	68.5	Average ISQ: 34
Average ISQ: 45	70.5	69.0	Average ISQ: 35
Average ISQ: 46	71.2	71.6	Average ISQ: 36
Average ISQ: 47	71.0	72.1	Average ISQ: 37
Average ISQ: 48	74.8	70.5	Average ISQ: 38

## Number of Success/Failures

<b>Number of Implants</b>	1450
---------------------------	------

<b>Success/Failures</b>	<b>Number</b>	<b>Percentage</b>
<b>Success</b>	1413	97,4%
<b>Failures</b>	37	2,6%

## Laser : Average ISQ at 6 weeks

<b>Global ISQ Average</b>	68.6
---------------------------	------

<b>ISQ Average</b>	<b>Same Period Length</b>
<b>Before Laser</b>	<b>After Laser</b>
From November 2000 to March 2002	From March 2002 to July 2004 (not complete)
67.7	69.1

<b>ISQ Average</b>	<b>Complete Period</b>
<b>Before Laser</b>	<b>After Laser</b>
From November 2000 to March 2002	From March 2002 to Now
67.7	69.0

## Success/Failures Before and After Laser Use (Complete Period)

<b>Total Number of Implants</b>	1450
---------------------------------	------

<b>From November 2000 to March 2002</b>	<b>Number</b>	<b>Percentage</b>
<b>Total of Concerned Implant</b>	437	30,13%
<b>Success</b>	425	97,3%
<b>Failures</b>	12	2,7%

<b>From March 2002 to Now</b>	<b>Number</b>	<b>Percentage</b>
<b>Total of Concerned Implant</b>	1013	69,9%
<b>Success</b>	988	97,5%
<b>Failures</b>	25	2,5%

## Success/Failures by Bone Volume Types

Number of Implants		1450		
Bone Volume Types	Number	Percentage	Global Percentage	
<b>A</b>	<b>Success</b>	473	98,1%	32,6%
	<b>Failures</b>	9	1,9%	0,6%
<b>B</b>	<b>Success</b>	667	97,4%	46%
	<b>Failures</b>	18	2,6%	1,2%
<b>C</b>	<b>Success</b>	265	96,7%	18,3%
	<b>Failures</b>	9	3,3%	0,7%
<b>D</b>	<b>Success</b>	5	100,0%	0,4%
	<b>Failures</b>	0	0,0%	0,0%

## Success/Failures by Gender

<b>Number of Implants</b>			1450
Patient Sex	Number	Percentage	Global Percentage
<b>Men</b>	<b>Success</b>	573	97,3%
	<b>Failures</b>	16	2,7%
<b>Women</b>	<b>Success</b>	840	97,6%
	<b>Failures</b>	21	2,4%

## Success/Failures by Bone Density Types

<b>Number of Implants</b>			1450
Bone Density Types	Number	Percentage	Global Percentage
<b>I</b>	<b>Success</b>	79	93,0%
	<b>Failures</b>	6	7,0%
<b>II</b>	<b>Success</b>	810	98,0%
	<b>Failures</b>	16	2,0%
<b>III</b>	<b>Success</b>	439	97,5%
	<b>Failures</b>	11	2,5%
<b>IV</b>	<b>Success</b>	80	96,4%
	<b>Failures</b>	3	3,6%

## Success/Failures by Implant Location

Number of Implants	1450
--------------------	------

Implant Location	Number	Percentage	Global Percentage
<b>18 to 11</b>			
Success	349	98,0%	24,1%
Failures	7	2,0%	0,5%
<b>21 to 28</b>			
Success	319	97,8%	22,0%
Failures	7	2,2%	0,5%
<b>31 to 38</b>			
Success	379	97,4%	26,2%
Failures	10	2,6%	0,7%
<b>48 to 41</b>			
Success	366	96,6%	25,2%
Failures	13	3,4%	0,8%

## Success/Failures by Implant Diameter

Number of Implants	1450
--------------------	------

Implant Diameter	Number	Percentage	Global Percentage
<b>4,00mm</b>			
Success	736	98,0%	50,8%
Failures	14	2,0%	1,0%
<b>4,75mm</b>			
Success	515	97,4%	35,5%
Failures	14	2,6%	1,0%
<b>6,00mm</b>			
Success	160	95,2%	11,0%
Failures	8	4,8%	0,5%

## Success/Failures by 4,00mm Diameter References

<b>Number of Implants</b>	<b>1450</b>
---------------------------	-------------

<b>References</b>		<b>Number</b>	<b>Percentage</b>	<b>Global Percentage</b>
<b>1020</b>	<b>Success</b>	29	90,6%	2,0%
	<b>Failures</b>	3	9,4%	0,2%
<b>1021</b>	<b>Success</b>	60	98,4%	4,1%
	<b>Failures</b>	1	1,6%	0,1%
<b>1022</b>	<b>Success</b>	138	98,6%	9,5%
	<b>Failures</b>	2	1,4%	0,1%
<b>1023</b>	<b>Success</b>	109	99,1%	7,5%
	<b>Failures</b>	1	0,9%	0,1%
<b>1024</b>	<b>Success</b>	220	97,8%	15,2%
	<b>Failures</b>	5	2,2%	0,4%
<b>1025</b>	<b>Success</b>	180	98,9%	12,4%
	<b>Failures</b>	2	1,1%	0,1%

## Success/Failures by 4,75mm Diameter References

Number of Implants		1450		
References		Number	Percentage	Global Percentage
1030	Success	45	95,7%	3,1%
	Failures	2	4,3%	0,1%
1031	Success	73	96,0%	5,0%
	Failures	3	4,0%	0,2%
1032	Success	187	97,9%	12,9%
	Failures	4	2,1%	0,3%
1033	Success	162	97,6%	11,2%
	Failures	4	2,4%	0,3%
1034	Success	48	98,0%	3,3%
	Failures	1	2,0%	0,1%

## Success/Failures by 6,00mm Diameter References

<b>Number of Implants</b>	1450
---------------------------	------

References	Number	Percentage	Global Percentage
<b>1040</b>			
Success	20	100,0%	1,4%
Failures	0	0,0%	0,0%
<b>1041</b>			
Success	31	96,9%	2,1%
Failures	1	3,1%	0,1%
<b>1042</b>			
Success	73	92,4%	5,0%
Failures	6	7,6%	0,4%
<b>1043</b>			
Success	32	96,9%	2,2%
Failures	1	3,1%	0,1%
<b>1044</b>			
Success	4	100,0%	0,3%
Failures	0	0,0%	0,0%

## Success/Failures by Immediate/Delayed Loading

<b>Number of Implants</b>	1450
---------------------------	------

Immediate/Delayed	Number	Percentage	Global Percentage
<b>Immediate</b>			
Success	810	99,6%	55,9%
Failures	13	0,4%	0,9%
<b>Delayed</b>			
Success	602	96,3%	41,5%
Failures	23	3,7%	1,6%

## Success/Failures by Immediate/Delayed Loading and Crest Types

<b>Number of Implants</b>	1450
---------------------------	------

Immediate Loading		Number	Percentage	Global Percentage
Edentulous Crest	Success	448	98,5%	30,9%
	Failures	7	1,5%	0,5%
Extraction Implantation	Success	361	98,4%	24,9%
	Failures	6	1,6%	0,4%

Delayed Loading		Number	Percentage	Global Percentage
Edentulous Crest	Success	499	96,1%	34,4%
	Failures	20	3,9%	1,4%
Extraction Implantation	Success	102	97,1%	7,0%
	Failures	3	2,9%	0,2%

# Success/Failures by Type of Prosthesis

<b>Number of Implants</b>	<b>1450</b>
---------------------------	-------------

Type of prosthesis	Number	Percentage	Global Percentage
<b>Single Crowns</b>			
<b>Success</b>	740	97,1%	51,0%
<b>Failures</b>	22	2,9%	1,4%
<b>Soldered Crowns</b>			
<b>Success</b>	86	97,7%	5,9%
<b>Failures</b>	2	2,3%	0,1%
<b>Bridges</b>			
<b>Success</b>	517	97,9%	35,6%
<b>Failures</b>	11	2,1%	0,7%

## Success/Failures by Types of Removable Prosthesis

<b>Number of Implants</b>		1450		
<b>Removable Prosthesis Type</b>	<b>Number</b>	<b>Percentage</b>	<b>Global Percentage</b>	
<b>Anchorage Bar</b>				
<b>Success</b>	18	100,0%	1,3%	
<b>Failures</b>	0	0,0%	0,0%	
<b>Attachment</b>				
<b>Success</b>	48	98,0%	3,3%	
<b>Failures</b>	1	2,0%	0,1%	